

# ON Semiconductor

## Is Now

# onsemi™

To learn more about onsemi™, please visit our website at  
[www.onsemi.com](http://www.onsemi.com)

---

**onsemi** and **onsemi** and other names, marks, and brands are registered and/or common law trademarks of Semiconductor Components Industries, LLC dba "**onsemi**" or its affiliates and/or subsidiaries in the United States and/or other countries. **onsemi** owns the rights to a number of patents, trademarks, copyrights, trade secrets, and other intellectual property. A listing of **onsemi** product/patent coverage may be accessed at [www.onsemi.com/site/pdf/Patent-Marking.pdf](http://www.onsemi.com/site/pdf/Patent-Marking.pdf). **onsemi** reserves the right to make changes at any time to any products or information herein, without notice. The information herein is provided "as-is" and **onsemi** makes no warranty, representation or guarantee regarding the accuracy of the information, product features, availability, functionality, or suitability of its products for any particular purpose, nor does **onsemi** assume any liability arising out of the application or use of any product or circuit, and specifically disclaims any and all liability, including without limitation special, consequential or incidental damages. Buyer is responsible for its products and applications using **onsemi** products, including compliance with all laws, regulations and safety requirements or standards, regardless of any support or applications information provided by **onsemi**. "Typical" parameters which may be provided in **onsemi** data sheets and/or specifications can and do vary in different applications and actual performance may vary over time. All operating parameters, including "Typicals" must be validated for each customer application by customer's technical experts. **onsemi** does not convey any license under any of its intellectual property rights nor the rights of others. **onsemi** products are not designed, intended, or authorized for use as a critical component in life support systems or any FDA Class 3 medical devices or medical devices with a same or similar classification in a foreign jurisdiction or any devices intended for implantation in the human body. Should Buyer purchase or use **onsemi** products for any such unintended or unauthorized application, Buyer shall indemnify and hold **onsemi** and its officers, employees, subsidiaries, affiliates, and distributors harmless against all claims, costs, damages, and expenses, and reasonable attorney fees arising out of, directly or indirectly, any claim of personal injury or death associated with such unintended or unauthorized use, even if such claim alleges that **onsemi** was negligent regarding the design or manufacture of the part. **onsemi** is an Equal Opportunity/Affirmative Action Employer. This literature is subject to all applicable copyright laws and is not for resale in any manner. Other names and brands may be claimed as the property of others.

## Adaptive Feedback Cancellation for the RHYTHM R3920 from ON Semiconductor



ON Semiconductor®

<http://onsemi.com>

### APPLICATION NOTE

#### INTRODUCTION

This information note describes the feedback cancellation feature provided in the latest digital hearing aid amplifiers for the Rhythm® R3920 from ON Semiconductor. Feedback cancellation differs from other feedback reduction strategies in that it does not reduce the forward gain of the hearing aid under normal operating conditions. Instead, the algorithm reduces feedback by formulating an internal estimate of the true hearing aid feedback signal and then subtracting this estimate from the microphone signal. If the internal estimate is accurate, then the actual feedback is cancelled and the hearing aid will not squeal.

Maintaining an accurate estimate of the true hearing aid feedback signal is essential to proper operation of the algorithm. Since the feedback signal changes over time, the algorithm must make continuous adjustments of the internal model to ensure accurate matching of the external feedback. In the feedback canceller from ON Semiconductor, these adjustments occur automatically and do not require probe

signals or other user interventions that might interfere with the operation of the hearing aid.

The latest feedback cancellation algorithm from ON Semiconductor features a new mechanism to increase adaptation speed while preserving audio quality. This mechanism automatically determines whether current feedback conditions require a large or small adjustment to the internal feedback-path model. Large adjustments are used to quickly eliminate acoustic feedback. When large adjustments are not needed, smaller adjustments are chosen to maximize audio quality.

The sections below describe feedback canceller operation in more detail highlighting factors that affect feedback canceller performance. The article concludes with a discussion of methods that can be used to optimize feedback canceller performance. These methods allow users of ON Semiconductor digital amplifiers to increase the performance of their products.

#### PRINCIPLE OF OPERATION

Adaptive Feedback Cancellation (AFC), from ON Semiconductor, is a true feedback cancellation (also known as *phase cancellation*) algorithm. The algorithm operates by subtracting an internal estimate of the hearing-aid feedback signal from the microphone signal. The internal feedback signal is obtained by passing the hearing-aid output signal through an internal, feedback-path model.

The feedback-path model in AFC is intended to match the impulse response, or time response, of the external feedback path. The feedback-path impulse response is the time signal that would be observed at the microphone output if an impulsive signal were produced by the hearing aid. As an example, the measured impulse response of a typical BTE hearing aid is shown in Figure 1-a. This plot shows an initial period of approximately 0.5 – 0.75 ms with little or no energy. This time window corresponds to the analog-to-digital and digital-to-analog converter delays as well as any acoustic delays due to transducers, tubing, sound propagation through the air, etc. Following the initial delay is the main part of the feedback-path response which, in this example, persists for more than 2 ms.

The net effect of the feedback canceller algorithm is to remove a certain portion of the feedback path impulse response. Since computational resources inside a hearing-aid signal processor are limited, only a portion of the total feedback path response can be cancelled. To achieve maximum effectiveness of the canceller, the cancelled part of the response should be that portion of the feedback path with the largest energy. This ensures that the net response has the lowest energy for the given length of canceller. For example, if the cancellation window is restricted to 1ms, then for the feedback path of Figure 1-a, the internal feedback model should look like the plot of Figure 1-b. When this is subtracted from the feedback-path response, the net feedback path is as shown in Figure 1-c. The corresponding frequency response is shown in Figure 2. This figure illustrates that the net effect of the feedback canceller in this example is to reduce the peak response of the feedback path by approximately 12 dB. Consequently, the feedback canceller in this example should provide approximately 12 dB of added stable gain (ASG) that is achieved without any impact on the forward gain of the hearing aid.

# AND9164/D

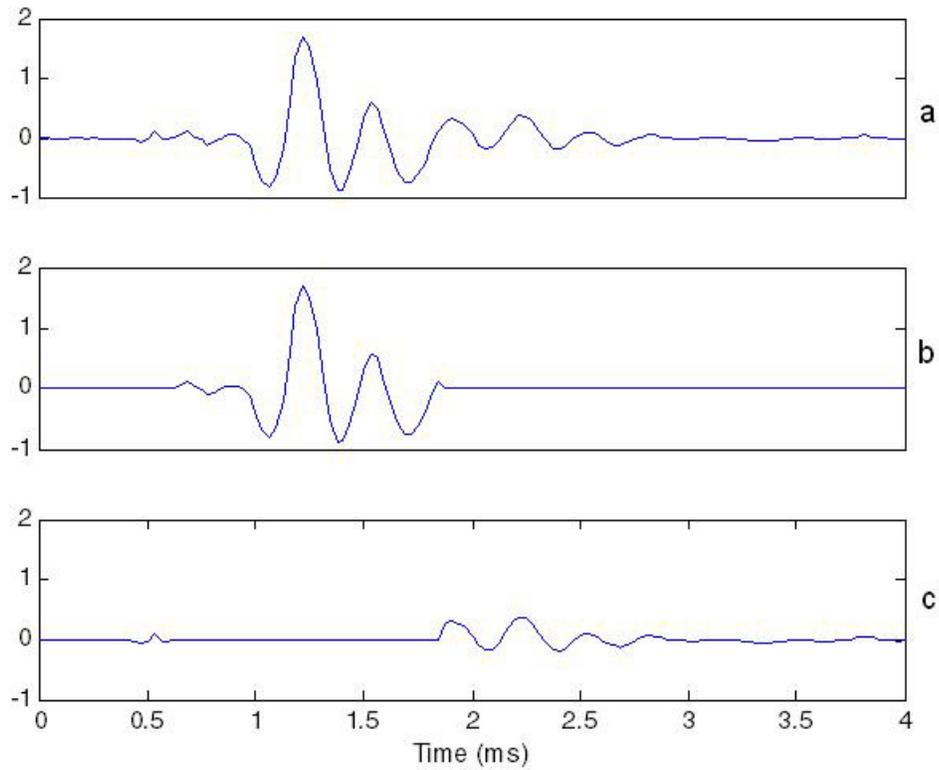


Figure 1. Measured impulse responses of a typical BTE: a) total external feedback path, b) internal feedback path of the feedback canceller, c) net effect including the effect of the feedback canceller

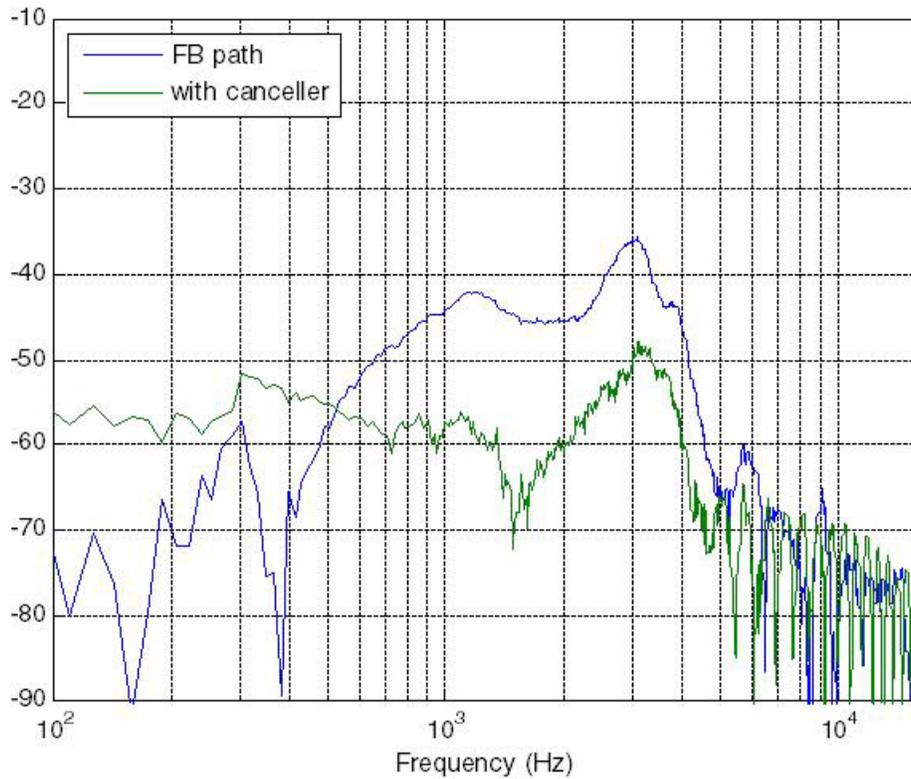


Figure 2. Change in feedback-path frequency response before and after the application of feedback canceller shown in Figure 1.

Unfortunately, the feedback path of a hearing aid is not constant. There is significant variability in the feedback paths of different hearing aids due to factors such as transducer selection, acoustic tubing, ear mold styles, etc. Furthermore, the feedback path of a specific hearing aid can also change with time due to user movements and environmental changes. If such changes are not reflected in the internal feedback path, then canceller performance can be significantly degraded.

To maintain optimal cancellation performance, AFC performs continuous updates of the internal feedback-path model. These updates occur automatically and utilize the existing hearing aid signals. No user intervention or special probe signals are required thus minimizing disruption for the hearing-aid wearer.

**OPTIMIZING PERFORMANCE**


Although conceptually simple, the performance of a feedback canceller algorithm in a hearing aid is influenced by many system-level factors. Feedback canceller performance is influenced by hearing-aid style, acoustic response, audio signal bandwidth, input-signal characteristics and interactions with other hearing-aid features. The following sections outline these influences and describe strategies for achieving optimum feedback canceller performance.

**Influence of Hearing Aid Style**

Hearing-aid styles influence feedback canceller performance because of the differences in feedback-path

acoustic delay. Simply put, the time required for sound to propagate from the receiver back to the microphone is different for an ITC compared to a BTE. The acoustic propagation time adds to the converter delays and appears at the start of the feedback path impulse response. This delays the onset of the maximum-energy segment of the feedback-path response. To maximize canceller performance, it must be compensated by the internal model.

As an example, the impulse responses of three different hearing aids (ITC, ITE and BTE) are shown in Figure 3. Note how the time onset of the impulse response occurs later for the hearing aid styles with the larger receiver/microphone separation.

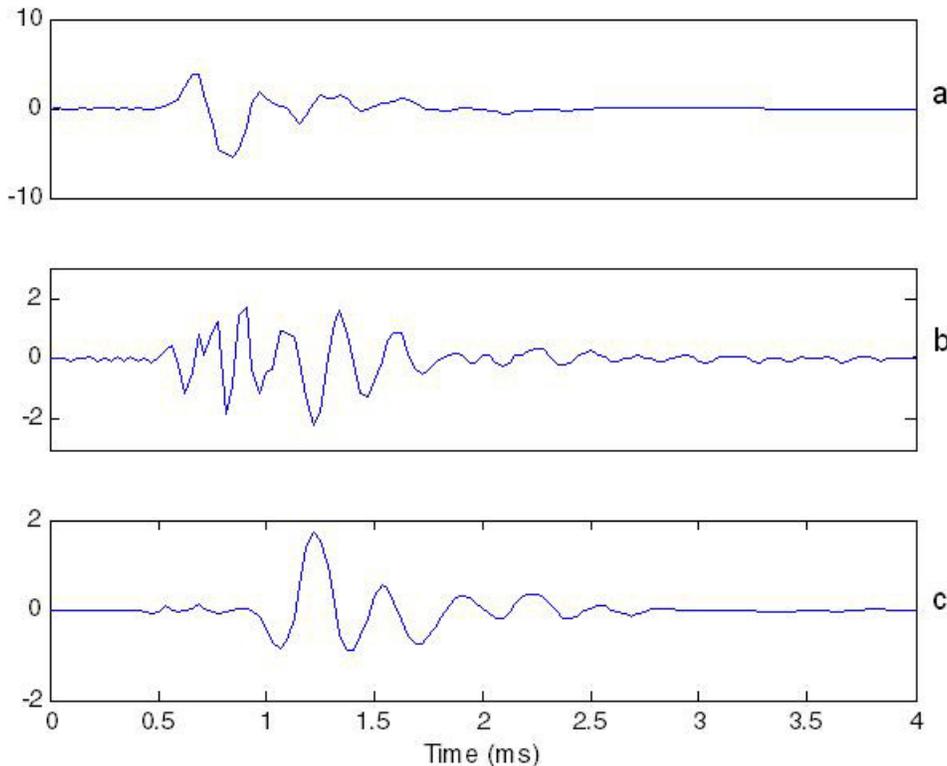

**Figure 3. Impulse responses from three different hearing aid styles: a) ITC, b) ITE, and c) BTE. Notice the increased initial delay for the larger styles.**

Generally, hearing aids with a larger physical separation between microphone and receiver (BTE, BTE-O and RITE)

required a longer acoustic predelay to achieve the highest ASG. Also, using an acoustic predelay that is not suited for the particular hearing-aid style can impair the achievable ASG.

It is best to perform an acoustic-pre-delay optimization procedure on any new hearing-aid style to which the ON Semiconductor feedback canceller is being applied. As a starting point, however, ON Semiconductor recommends

the acoustic pre-delay settings shown in Table 1. Note that the acoustic pre-delay is different for the 16 kHz and 32 kHz sampling frequencies. In fact, the acoustic pre-delays for 32 kHz are approximately double of those for 16 kHz.

**Table 1. RECOMMENDED ACOUSTIC PRE-DELAY VALUES FOR DIFFERENT HEARING-AID STYLES**

Hearing Aid Style	Recommend Acoustic Predelay		Receiver to Mic Distance (cm)
	16 kHz	32 kHz	
CIC	7	14	2
ITC	8	15	2.5
ITE	8	15	3.5
RITE	13	25	8
BTE	18	35	14


An adjustable *acoustic predelay* parameter (FBC AcouDelay) accommodates different hearing aid styles. This parameter allows the time location of the feedback canceller operation to be optimized for different hearing aid styles without adding any computational complexity or power consumption to the hearing aid circuit.

The FBC AcouDelay parameter can be found on the advanced features tab in ON Semiconductor Interactive Data Sheet (IDS) software, as shown in Figure 4. Notice that

the recommended settings given in Table 1 are provided on the screen.

**Adjusting Adaptation Speed**

The flexibility of the feedback cancellation algorithm has also been enhanced by providing tunable parameters that control the size of the feedback-path model adjustments under all conditions. The feedback canceller is enabled using the advanced features tab in ON Semiconductor IDS software, as shown in Figure 4.

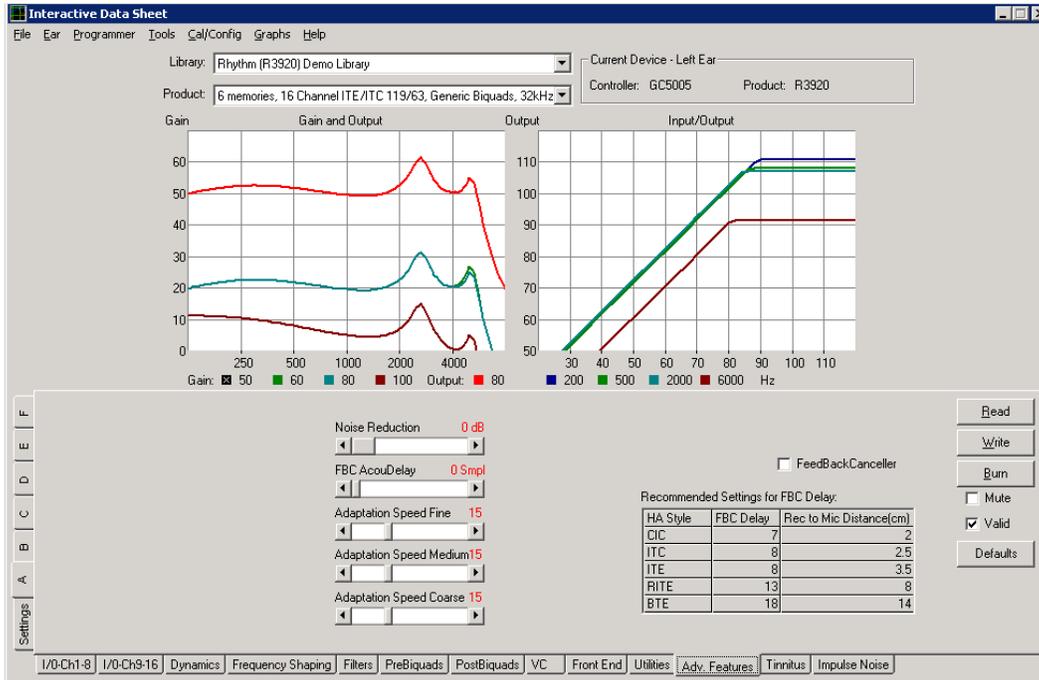

**Figure 4. Advanced Features Tab in ON Semiconductor Interactive Data Sheet (IDS)**

The three parameters below provide a “gear shifting” mechanism that reduces the size of the updates as the adaptation nears completion. During adaptation, the

algorithm is instantaneously changing its speed. This dynamic adjustment allows the feedback canceller behavior to be tuned to specific devices or circumstances.

# AND9164/D

Adaptation Speed Fine	Controls the size of the smallest adjustments
Adaptation Speed Medium	Used under all other conditions that do not warrant fine or coarse control
Adaptation Speed Coarse	Controls the size of the largest adjustments

## Feedback Canceller Characterization


It is possible to reduce the convergence time by decreasing the adaptation speed parameters. For illustrative purposes, the Table 2 shows the resulting adaptation speed with all parameters set to the same value. Actual adaptation speed may vary depending on acoustic conditions, but the trend is representative of what can be expected.

**Table 2. INFLUENCE OF ADAPTATION SPEED PARAMETERS ON CONVERGENCE TIME**

	Adaptation Speed Parameters (same values for Coarse, Medium and Fine)							
	14	15	16	17	18	19	20	21
Average Convergence Time(s)	1.9	2.1	2.1	2.4	2.8	5.5	10.0	11.9

There is a trend that the signal distortion increases with faster adaptation. In other words, although decreasing the adaptation speed parameters results in faster convergence, as shown in Table 2, decreasing these may also result in additional non-linear distortion (also known as entrainment) with tonal inputs. The logarithmic spectral difference of the

output signals (feedback canceller on vs. feedback canceller off) was used to describe the amount of entrainment in a particularly challenging input signal. Figure 5 illustrates how the resulting distortion decreases as the adaptation speed parameters increase.

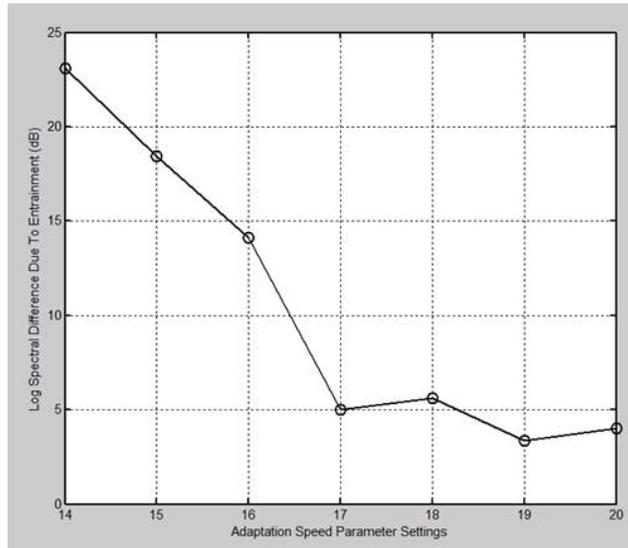

**Figure 5. Distortion in Relation to Adaptation Speed**

Knowing this tradeoff, one may choose the adaptation speed parameter settings that give the desired performance. For instance, it is possible to use different settings for each of the coarse, medium and fine step sizes. This takes

advantage of the “gear shifting” mechanism described above and permits a better trade-off between convergence time and entrainment. Two such examples and the corresponding adaptation speeds are shown Table 3.


**Table 3. EXAMPLES OF OPTIMIZING CONVERGENCE TIME AND ENTRAINMENT**

Adaptation Speed Parameters	Coarse: 15 Medium: 16 Fine: 17	Coarse: 18 Medium: 19 Fine: 20
Log Spectral Difference (dB)	14.4	3.6
Average Convergence Time (s)	1.88	10.66

### Acoustic Response

Even for hearing aids of the same style, significant performance differences can be observed. The reason for this is that the feedback canceller effectively removes a limited segment of the impulse (time) response of the feedback path. For feedback paths with a short time duration, the feedback canceller can remove a large proportion of the total energy; for feedback paths with a long time duration, the energy removed is a smaller proportion of the total, resulting in a lower ASG.

To illustrate, the impulse responses of two ITE-style hearing aids are shown in Figure 6. Since the physical sizes of the two devices are similar, they possess similar onset times; however, the response shown in the top graph has a much longer time duration than the response shown in the bottom graph. The feedback-path frequency responses for the same systems are shown in Figure 7. It can be seen in Figure 7 that the first ITE with the longer time duration impulse response is manifested as a very “peaky” frequency response plot.

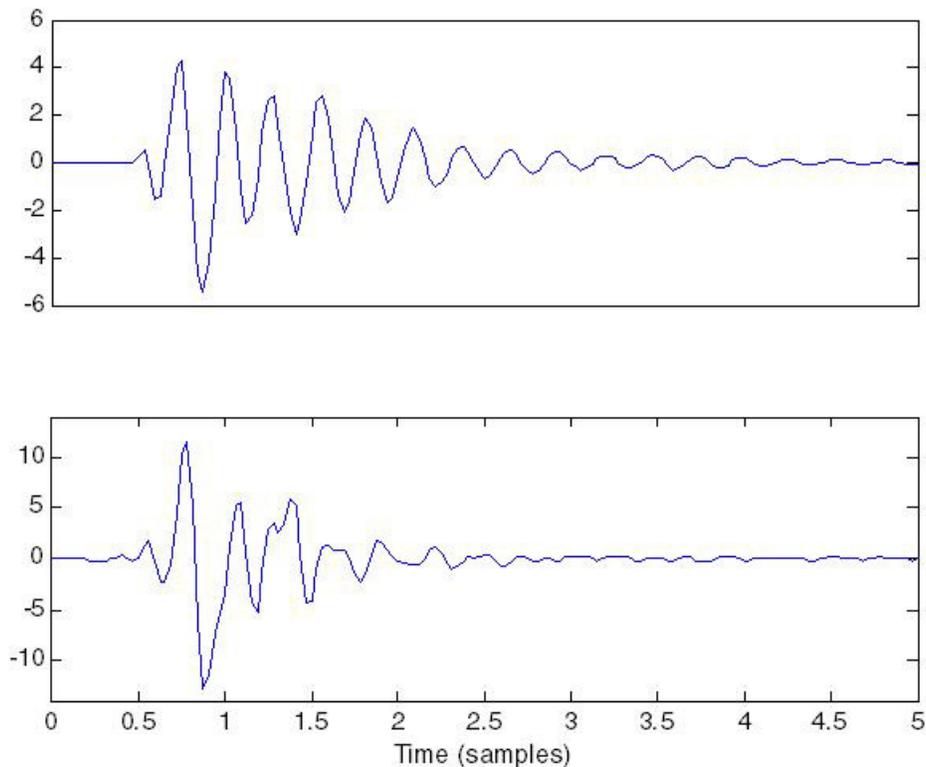


**Figure 6. Impulse response for two different ITE hearing aids. Response shown in the top graph has a longer time duration and will likely result in less ASG compared to the response shown in the bottom graph.**

There are many factors that can contribute to such peaky feedback path responses: sharp resonances in transducer responses (most notably receivers); acoustic tube resonances and vibration coupling problems. All of these

can lead to peaky feedback responses and, ultimately, to poor feedback canceller performance. For optimum feedback canceller performance, such resonances should be eliminated from the system design, as much as possible.

## AND9164/D

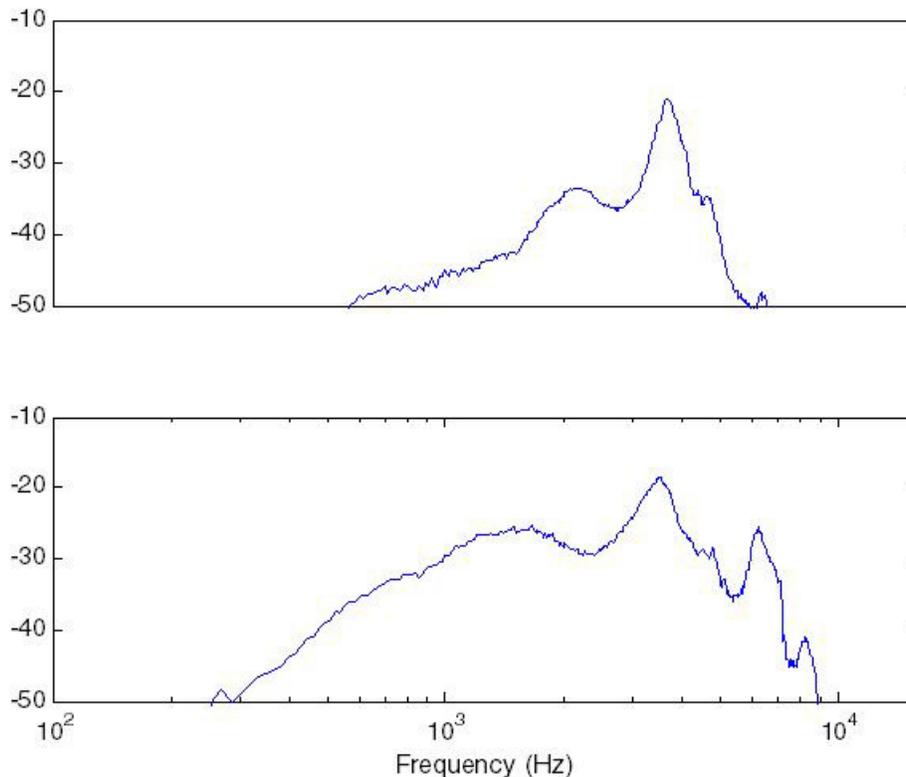

**Figure 7. Frequency response for the two ITE hearing aids shown in Figure 6. Response shown in the top graph has a sharper response peak and will likely result in less ASG compared to the response shown in the bottom graph.**

### Audio Signal Bandwidth

Digital amplifiers from ON Semiconductor offer the flexibility of operating at an audio sample rate of either 16 or 32 kHz, providing audio bandwidths of 8 and 16 kHz, respectively. While the feedback canceller supports both sample rates, some slight differences in performance might be observed.

As described above, the internal feedback path model is only capable of representing a finite time segment of the actual feedback-path response. This finite-time segment consists of a fixed number of audio samples regardless of the sample rate. As the sample rate is lowered, the sample period increases and, for a fixed number of samples, the canceller time window is longer. A longer canceller time window means more of the feedback path can be internally modelled and cancelled, as in Figure 1. This generally leads to better feedback canceller performance (higher ASG).

In general, it is reasonable to expect higher ASG for a given hearing aid when the DSP is run at a lower sample rate. Due to acoustical variances between different hearing aid designs (as shown in Figures 6 and 7), however, the same benefits may not be observed in all cases.

### Input Signal Characteristics

In order to minimize disruption for the hearing-aid wearer, feedback canceller updates are performed while the hearing aid is operating. Unfortunately, the adaptation algorithm that is used to update the internal model is sensitive to the

microphone-signal characteristics. The adaptation algorithm is ideally trained using non-tonal or noise-like signals. Signals that are tonal, or music-like, can cause maladjustment of the internal feedback-path model leading to renewed feedback or other audible artefacts. This maladjustment phenomenon is sometimes referred to as entrainment.

AFC is specifically designed to minimize entrainment artefacts. A sophisticated signal analysis scheme coordinates the adaptation control to permit rapid identification of the feedback path during ideal conditions but to slow adaptation when tonal signals or music are detected. The result is a substantial reduction of entrainment artefacts for music and tones when compared to previous products.

### Interaction with Other Hearing Aid Features

An adaptive feedback canceller requires a certain amount of time to adjust itself to any changes in the external environment. While feedback cancellers typically respond to slow variations that occur naturally during hearing aid usage, advanced hearing-aid features may cause much more rapid changes to occur. For example, an adaptive directional microphone can cause a sudden change in the external feedback path since it is rapidly changing the microphone's directional response. Similarly, an adaptive noise reduction algorithm may cause a very rapid gain change if the

## AND9164/D

classification of a frequency band changes from noise to speech.

If a sudden change occurs in the external feedback path there will be a temporary mismatch with the internal feedback-path model, even though the hearing-aid gain did not change. If the gain is sufficiently high, this may lead to a sudden, but temporary feedback event.

Also if the hearing-aid gain changes too rapidly, even though the external feedback path did not change, a small maladjustment of the internal model can be amplified and can lead to a temporary feedback event.

In light of these potential interactions, AFC was specifically designed to coexist with other advanced hearing aid features. Improvements in adaptation accuracy, reduced entrainment together with increased coordination ensure minimal disturbances of feedback canceller operation.

Despite the improvements in AFC, the unmatched flexibility offered in ON Semiconductor digital hybrids may still result in situations where feedback canceller performance may be impaired. One such case is with the application of the in-channel squelch algorithm.

In certain advanced hybrids, ON Semiconductor offers the ability to apply squelch independently in each compression channel. Experience has shown, however, that having drastically different squelch settings can lead to problems with feedback canceller performance. ON Semiconductor, therefore, recommends that the squelch ratio be restricted to 1:2 and be enabled in all channels when also using adaptive feedback cancellation. Furthermore, the squelch thresholds should be set higher than the microphone noise floor.

### SUMMARY

AFC from ON Semiconductor is a true feedback cancellation algorithm, removing acoustic feedback without compromising the forward gain of the hearing aid. It allows higher forward gain while providing immunity to tonal

inputs and minimal entrainment artefacts. The algorithm is designed to coexist with other adaptive hearing-aid features and allows hearing-instrument designers the flexibility to optimize performance for their individual products.

RHYTHM is a trademark of Semiconductor Components Industries, LLC (SCILLC).

ON Semiconductor and  are registered trademarks of Semiconductor Components Industries, LLC (SCILLC). SCILLC owns the rights to a number of patents, trademarks, copyrights, trade secrets, and other intellectual property. A listing of SCILLC's product/patent coverage may be accessed at [www.onsemi.com/site/pdf/Patent-Marking.pdf](http://www.onsemi.com/site/pdf/Patent-Marking.pdf). SCILLC reserves the right to make changes without further notice to any products herein. SCILLC makes no warranty, representation or guarantee regarding the suitability of its products for any particular purpose, nor does SCILLC assume any liability arising out of the application or use of any product or circuit, and specifically disclaims any and all liability, including without limitation special, consequential or incidental damages. "Typical" parameters which may be provided in SCILLC data sheets and/or specifications can and do vary in different applications and actual performance may vary over time. All operating parameters, including "Typicals" must be validated for each customer application by customer's technical experts. SCILLC does not convey any license under its patent rights nor the rights of others. SCILLC products are not designed, intended, or authorized for use as components in systems intended for surgical implant into the body, or other applications intended to support or sustain life, or for any other application in which the failure of the SCILLC product could create a situation where personal injury or death may occur. Should Buyer purchase or use SCILLC products for any such unintended or unauthorized application, Buyer shall indemnify and hold SCILLC and its officers, employees, subsidiaries, affiliates, and distributors harmless against all claims, costs, damages, and expenses, and reasonable attorney fees arising out of, directly or indirectly, any claim of personal injury or death associated with such unintended or unauthorized use, even if such claim alleges that SCILLC was negligent regarding the design or manufacture of the part. SCILLC is an Equal Opportunity/Affirmative Action Employer. This literature is subject to all applicable copyright laws and is not for resale in any manner.

### PUBLICATION ORDERING INFORMATION

#### LITERATURE FULFILLMENT:

Literature Distribution Center for ON Semiconductor  
P.O. Box 5163, Denver, Colorado 80217 USA  
Phone: 303-675-2175 or 800-344-3860 Toll Free USA/Canada  
Fax: 303-675-2176 or 800-344-3867 Toll Free USA/Canada  
Email: [orderlit@onsemi.com](mailto:orderlit@onsemi.com)

**N. American Technical Support:** 800-282-9855 Toll Free  
USA/Canada  
**Europe, Middle East and Africa Technical Support:**  
Phone: 421 33 790 2910  
**Japan Customer Focus Center**  
Phone: 81-3-5817-1050

**ON Semiconductor Website:** [www.onsemi.com](http://www.onsemi.com)

**Order Literature:** <http://www.onsemi.com/orderlit>

For additional information, please contact your local Sales Representative